


# Sizing chart

## circaid® reduction kit lower leg

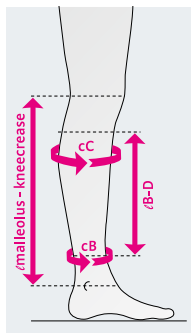

Circumferences in cm	standard	wide
cC	< 65	< 85
cB	< 55	< 65

Actual garment length in cm	short	standard	long
∟ B - D	30	35	40


  

Length measurement in cm	short	standard	long
∟ malleolus - knee crease	< 32	32 – 36	> 36

All products contain the Built-In-Tension System card (BTS).

# Sizing chart

## circaid® reduction kit knee

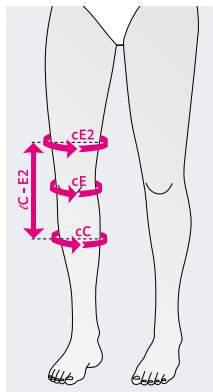


Circumferences in cm	standard
cE2*	< 110
cE	< 102
cC	< 80
Length in cm	standard
lC - E2	30

\* cE2 should not be more than 30 cm bigger than cC.  
cE2 is 10 cm above the patella

# Sizing chart

## circaid® reduction kit upper leg

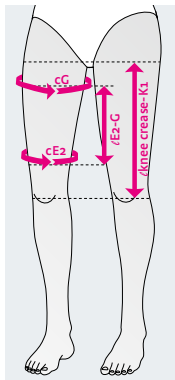

Circumferences in cm	standard	wide	
cG*	< 80	< 125	
cE2**	< 65	< 110	
Actual garment length in cm	short	standard	long
l E2 - G	30	35	40
Length measurement in cm	short	standard	long
l knee crease - K1	< 37	37 - 41	> 41


\* cG should not be 30 cm bigger than cE2

\*\* cE2 is 10 cm above the patella

All products contain the Built-In-Tension System card (BTS).

# Sizing chart

## circaid® reduction kit arm

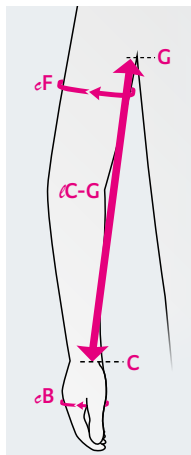

circaid  
reduction  
kit arm

Circumferences in cm	standard	wide	extrawide
cF	< 50	50 – 65	65 – 80
cB*	< 30	< 30	< 30
Length in cm	standard	long	
∠ C - G	< 45	> 45	

All products contain the Built-In-Tension System card (BTS).


\* If cB is > 30 cm, the enclosed circaid customizable hand wrap will not fit. But the circaid reduction kit arm garment may still be used. We recommend to use a circaid juxtafit made-to-measure hand option. For detailed information please see our product catalogue lymphology (Art.No. 0097D84).

### Note

Comes with one pair of undersleeves and a circaid customizable hand wrap.

# Sizing chart

## circaid® juxtafit® premium and essentials lower leg

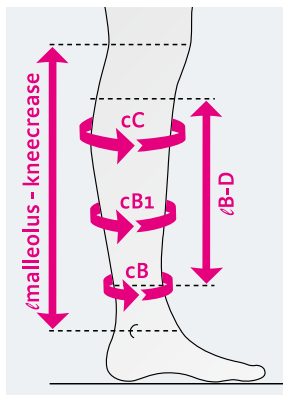


Circumferences in cm	S	M	M - X	L
cC	30 - 38	36 - 43	43 - 50	41 - 48
cB1	20 - 28	25 - 33	33 - 40	30 - 38
cB	15 - 23	20 - 28	20 - 28	25 - 33
	L - X	XL	XL - X	XXL
cC	48 - 56	41 - 51	51 - 61	53 - 64
cB1	38 - 46	33 - 41	41 - 51	43 - 53
cB	25 - 33	30 - 38	30 - 38	33 - 41

Actual garment length in cm	short	long
∠ B - D	28	36
Sizing measurement in cm	short	long
∠ malleolus - knee crease	< 36	> 36

All garments contain the circaid Built-In-Pressure System card (BPS).

# Sizing chart

## circaid® juxtafit® essentials upper leg with knee

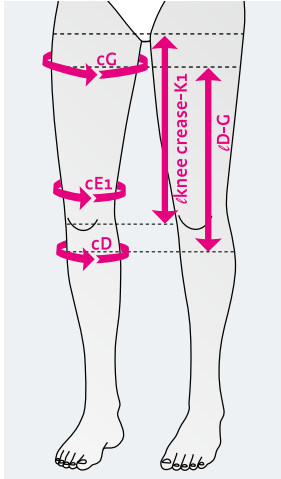

Circumferences in cm	XXS	XS	S	M	L	XL
cG	40-52	46-62	54-70	60-80	69-90	77-101
cE1*	32-42	37-48	42-55	50-65	58-76	66-87
cD	23-30	27-35	31-41	35-46	40-52	44-58
Actual garment length in cm		extra short	short	long		
lD-G		35**	45**	55**		
Sizing measurement in cm		extra short	short	long		
l knee crease - K1		< 25	25-35	> 35		


All garments contain the circaid Built-In-Pressure System card (BPS).

\* cE1 is 5 cm above the patella

\*\* Product length can be shortened by 5 cm

# Sizing chart

## circaid® juxtafit® essentials arm

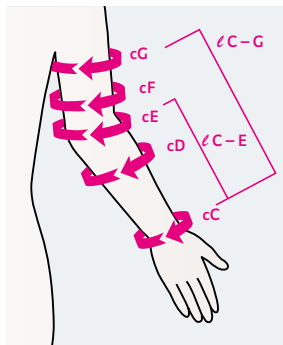


Circumferences in cm	1	2	3	4	5	6
cG	28–32	30–34	32–37	35–40	38–44	42–48
cF	26–30	28–32	30–35	33–38	36–41	39–45
cE	25–28	27–30	29–32	31–34	33–36	36–40
cD	23–26	25–28	27–30	29–32	31–34	33–37
cC	15–17	17–19	19–21	21–23	23–25	25–28
<b>Actual garment length in cm</b>		<b>extra short</b>		<b>short</b>		<b>long</b>
lC–G		38		43		48
<b>Sizing measurement in cm</b>		<b>extra short</b>		<b>short</b>		<b>long</b>
lC–E		< 22		22–25		> 25

All garments contain the circaid Built-In-Pressure System card (BPS).

# Sizing chart

circaid® juxtacures®

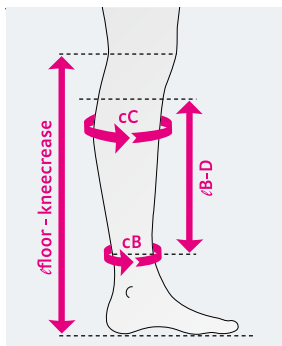

## Circumferences in cm

Adjustable to patient-individual circumferences.

## Minimum and maximum circumference measurements in cm

	minimum	maximum
cB	19	42
cC	25	64

## Actual garment length in cm

	short	standard	long
∠ B - D	28	33	38

## Sizing measurement in cm

	short	standard	long
∠ floor - knee crease	< 44	44 - 49	> 49

All garments contain the circaid Built-In-Pressure System card (BPS).


## Note

Comes with one pair of undersleeves and one pair of compression anklets.



# Sizing chart


circaid® juxtalite® lower leg

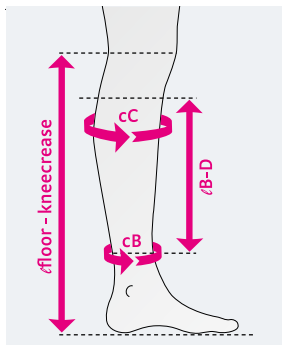

Circumferences in cm	S	M	M - X	L
cC	26 - 36	33 - 44	44 - 54	41 - 51
cB	19 - 22	22 - 27	22 - 27	25 - 32
	L - X	XL	XL - X	XXL
cC	51 - 61	37 - 48	48 - 61	53 - 64
cB	25 - 32	30 - 37	30 - 37	35 - 42
<b>Actual garment length in cm</b>	<b>short</b>	<b>long</b>		
∠ B - D	28	33		
<b>Sizing measurement in cm</b>	<b>short</b>	<b>long</b>		
∠ floor - kneecrease	< 44	> 44		

All garments contain the circaid Built-In-Pressure System card (BPS).

## Sizing chart

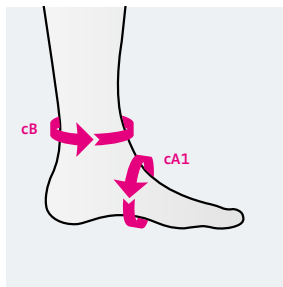

### Note


Circumferences in cm.

Circumference cB is around the ankle and cA1 around the arch.

<b>circaid compression anklet</b>	<b>standard</b>	<b>large</b>
<b>cB</b>	19–27	25–50
<b>cA1</b>	19–36	22–41

<b>circaid pac band</b>	<b>standard</b>	<b>large</b>	<b>extra large</b>
<b>cB</b>	19–27	25–50	45–56
<b>cA1</b>	19–36	22–41	35–44

## Sizing chart

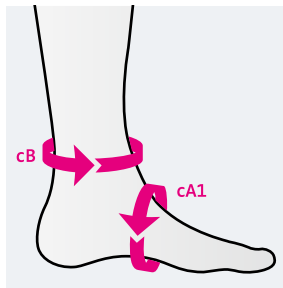

### Note

Circumferences in cm.

Circumference cB is around the ankle and cA1 around the arch.


### circaid customizable interlocking ankle foot wrap

cB	< 56
cA1	< 44

### circaid single band ankle foot wrap

Width	7,6 cm
Length	203 cm

## Sizing chart

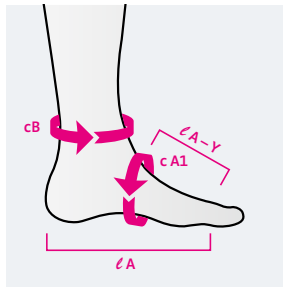

### Note

Circumferences in cm.


Circumference cB is around the ankle and cA1 around the arch.

<b>circaid juxtafit premium interlocking ankle foot wrap</b>	<b>S</b>	<b>M</b>	<b>L</b>
lA-Y	>9	>9	>9
cB	<36	<36	<41
cA1	19-24	24-29	29-34
lA	13-25	13-25	13-25

<b>circaid juxtafit premium ankle foot wrap</b>	<b>S</b>	<b>M</b>	<b>L</b>
lA-Y	>9	>9	>9
cB	<36	<36	<41
cA1	19-24	24-29	29-34
lA	>16	>16	>16

## Sizing chart

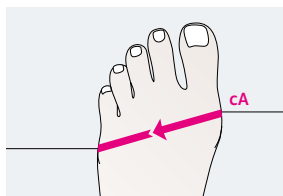


### Note

Circumferences in cm.

circaid juxtalite ankle foot wrap	S	M
cA1	19–26	24–31

circaid toe cap	2	3	4	5	6
cA	20–22	22–24	24–26	26–28	28–30

## Sizing chart

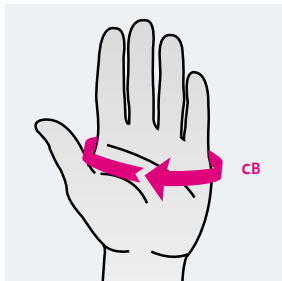

### Note

Circumferences in cm.

For made-to-measure circaid juxtafit hand options please see our product catalogue lymphology (Art.No. 0097D84).

### circaid customizable hand wrap


cC < 30

cB < 30

### circaid juxtafit® essentials hand wrap

	XS	S	M	L	XL
cB Palm	14 – 17	17 – 20	20 – 23	23 – 26	26 – 29

## Sizing chart

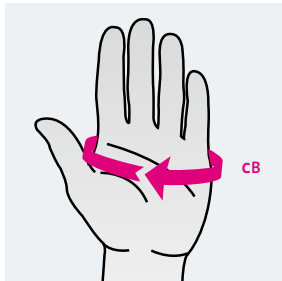

### Note

Circumferences in cm.

### circaid glove

	2	3	4	5	6
cB Palm	18–21	20–23	22–25	24–27	26–29